




How to use the Bellus3D FaceApp: A Step-By-Step Process

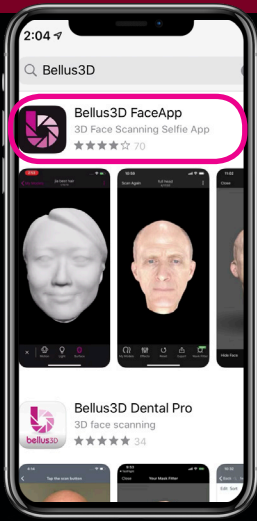

Go to your iTunes App Store on your mobile device and download the "Bellus3D FaceApp".

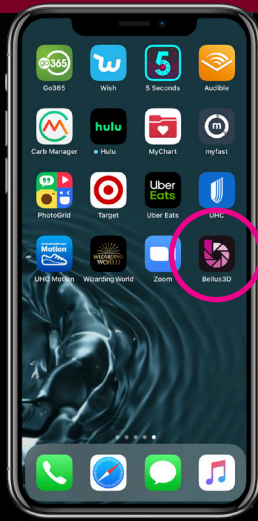

Once downloaded, open the app on your mobile device. Create an account with a username and password.

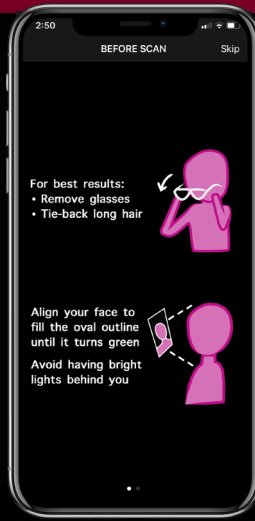

The app will then show you how to take a proper face scan.



After the short tutorial, it will prompt you take a scan of your own.




Hold up your phone and follow the audio instructions.




The app will process your movements to create a 3D scan of your face.



The scan of your face should show up on the home screen. Next, click "Mask Fitter" in your workspace.

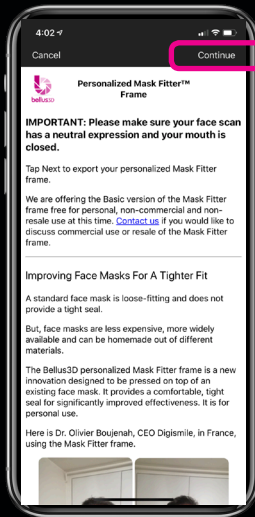


Click "Continue" to view your virtual mask fitter.

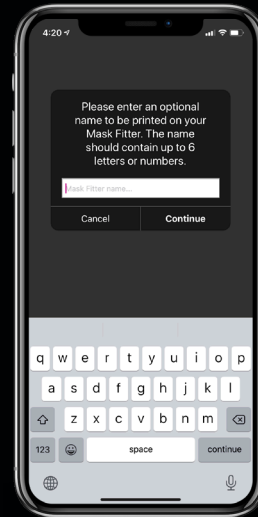

Personalize your mask fitter.



The app will create another 3D scan of your face. This time, with a mask fitter.

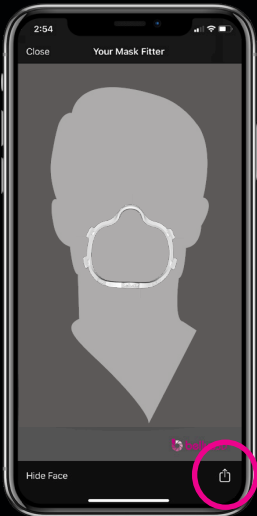

To export this .STL file, click on the icon in the bottom right corner.



Different exporting options will show on your phone. You can email directly to digital@ottawadentallab.com.



**OTTAWA DENTAL
LABORATORY**

Make sure to have the .STL file readily available when you place your order of mask fitters. If you have any questions or if you need assistance please call 800-851-8239 or email customerservice@ottawadentallab.com.

We look forward to working with you and providing the proper PPE to your office!

Compatible Devices: iPhone X, Xs, Xs Max, XR, 11, 11 Pro, 11 Pro Max

Questions?

Please call 800-851-8239 for assistance.

Ottawa Dental Laboratory
1304 Starfire Dr. • Ottawa, IL 61350